


## Case Study

# Use of the INnate™ Intramedullary Threaded Nail for Comminuted Fractures of the Third and Fourth Metacarpals

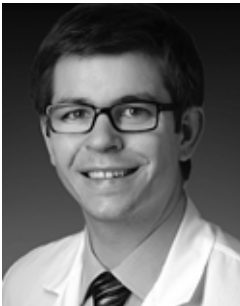

## Derek Masden, MD, FACS

Vice Chairman of Plastic Surgery, MedStar Washington Hospital Center Plastic Surgery / Hand Surgery, MedStar Washington Hospital Center Assistant Clinical Professor  
Department of Plastic Surgery, MedStar Georgetown University Hospital  
Washington, D.C.

Dr. Masden is a board certified plastic surgeon. After graduating from the University of Rochester School of Medicine, he did his plastic surgery residency at Georgetown University. He is currently the chief of plastic surgery at Luminis Health in Washington, D.C.

## Case Presentation

Patient was a 59-year-old female who suffered a gunshot wound to her right hand during protests in Washington, DC. For her bones to heal properly, she required a rigid framework to prevent collapse and significant shortening of her metacarpals. Patient had highly comminuted midshaft and neck fractures to her third and fourth metacarpals. Due to concerns of shortening, compression screws or K-wire fixation were eliminated as options. To achieve stable fixation, an intramedullary nail that spanned through the entire metacarpal was chosen.

## Preop Plan

Dr. Masden planned to proceed with INnate™ Intramedullary Threaded Nail to achieve a stable construct, allowing early range of motion (ROM) and minimizing the cost and inconvenience of a potential return to the operating room. He felt that due to its non-compressive nature, the INnate nail would effectively restore metacarpal length, which would be determined intraoperatively by comparing the fractured bone to the adjacent metacarpals, as well as utilizing an AP radiograph of the contralateral hand.

## Operative Findings and Approach

After exploring the wounds, Dr. Masden confirmed that the flexor and extensor tendons were intact. Once both fractures were reduced, he used the INnate depth gauge to determine that a 4.5 mm x 55 mm INnate nail was needed for the third metacarpal and a 3.6 mm x 45 mm nail was needed for the fourth metacarpal. Dr. Masden made a 2 mm incision on the dorsal third of the metacarpal head and inserted the provided K-wire across the fracture site under fluoroscope. He then used the cannulated drill to drill over the guidewire and threaded the cannulated INnate nail until the head was beneath the articular cartilage. The nail achieved proximal purchase at the isthmus level, and was left subchondral, maximizing distal purchase. Full restoration of height was achieved. Total operating time was 89 minutes, which included exploratory/adjunctive surgery and implanting.


## Follow-up

In less than two weeks postop, the patient achieved 75% of her full range of motion, which Dr. Masden feels would have been difficult to replicate with any other fixation techniques or implants.

## Discussion

Dr. Masden's approach allowed the patient to experience excellent outcomes and full functionality that would not have been possible with alternative solutions. The patient's early mobilization allowed for earlier physical therapy. Dr. Masden also reduced his costs by minimizing his time in the operating room due to the ease of use with INnate.

## Preoperative

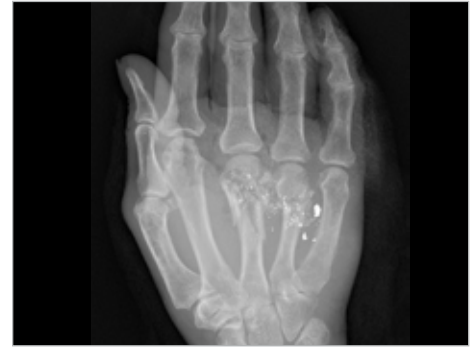

## Postoperative



These materials contain information about products that may or may not be available in any particular country or may be available under different trademarks in different countries. The products may be approved or cleared by governmental regulatory organizations for sale or use with different indications or restrictions in different countries. Products may not be approved for use in all countries. Nothing contained on these materials should be construed as a promotion or solicitation for any product or for the use of any product in a particular way which is not authorized under the laws and regulations of the country where the reader is located. Specific questions physicians may have about the availability and use of the products described on these materials should be directed to their particular authorized Acumed distributor. Specific questions patients may have about the use of the products described in these materials or the appropriateness for their own conditions should be directed to their own physician.

